NAGGURA SKIN

The best skin for your treatment table

PHYSIO (



White ngg 51



Graphite grey ngg 52

ADVANCED VYNIL ARTIFICIAL LEATHER

ITEM REFERENCE	ngg51 (white) / ngg 52 (graphite grey)		
ABRASION RESISTANCE	>250.000 cycles (Martindale scale)		
TENSILE PROPERTIES	Tension load, warp-wise N/3cm	Standard=>120	Testing result 495
	Tension load, acrross-warp N/3cm	Standard=>70	Testing result 300
	Elongation at break, warp-wise %	Standard =>20	Testing result 93
	Elongation at break, accross-warp %	Standard=>120	Testing result 165
TEARING STRENGTH N/3cm	Warp-Wise N	Standard=>10	Testing result 38
	Across warp N	Standard=>9	Testing result 28
PEEL STRENGTH	Warp-Wise N	Standard=>10	Testing result 14
COLD TOLERANCE	Cold crack resistance (-23°C, 5 min.)		
FLAME RETARDANT	NFP92-503/M2(non-flammable)		

DAILY CARE & CLEANING

* Clean at least once a week to maintain the coating clean and prevent accumulation of dirt and contaminants. This will extend the lifespan of your upholstery

* Clean the entire surface using neutral PH liquid soap and water in circular motions. To do so, use a white clean cloth soaked in the cleansing solution and wring out the wet cloth until all water is removed.

*Dry the surface using a white and clean cloth.

*Never use solvents, detergents and/or cleaning products that are not pH neutral.

*Never use cleaning tools made of plastic, abrasive sponges or hard-bristle brishes.