



# TheWellness Company

## Contents:

Ultrasound & Galvanic 2

High Frequency & Vacuum & spray 3

Rotary brushes & exfoliating spatule 4

Diamond dermabrasion & Microcrystal 5

Pressotherapy 6/7

Thermotherapy & spot removal 8

Body massager & Cold and hot hammer 9

Radiofrequency & High conductivity gloves and BIO LIFT 10

Electrostimulation 11/12

Steamers 13/14

Sterilisers 15/16

Spare parts 17/20

## Guidelines for Beauty treatments

# Aesthetic treatments through Weelko professional instruments

**Weelko** is a new concept that transcends wellness products. It is an attitude and a way of understanding **well-being** in a global scale; a way of caring for the smallest details in order to care for people. To make those enjoying our equipment and furniture feel comfortable in the broadest sense of the Word.

With this aim, we develop specific lines for **Beauty**, **SPA&wellness**, **Physiotherapy** and **Podiatry field**. All of them designed to guarantee the maximum well-being to professionals and to their patients without forgetting those specialists in distribution who trust in the quality of our service. From Europe and crossing its borders.

Adjustable heights, intuitive methods of use, ergonomic lines, pleasant textures, minimal storage spaces, easy-to-understand instructions and simple assemblies are, among others, factors that make Weelko products practical, functional and, above all, comfortable. The essential is provided.




The superfluous is left aside. **Technologically** advanced products designed to work in a more pleasant way. The easier, **the better**.



**Welcome to Weelko, the wellness company that makes it easy.**  
**Come in, feel good and relax**


	TREATMENT BENEFITS	REMARKS	PRODUCT	ACCESSORIES
<b>ULTRA-SOUND</b>  F-312A F-801C F-815 F-812 F-330 F-332 F-337 P-01	<ul style="list-style-type: none"> <li>Helps to smooth and soften fine lines and wrinkles.</li> <li>Improves blood circulation.</li> <li>Increases cellular regeneration.</li> <li>Has a healing effect on wounds and inflamed tissues.</li> <li>Increases the moisture retention of the skin.</li> <li>Improves <i>acneic</i> skin and reduces pore size.</li> <li>Gives the skin a healthy glow.</li> <li>Pushes products into the skin.</li> <li>Also indicated for physiotherapy and massage treatments.</li> </ul>	<ul style="list-style-type: none"> <li><b>Continuous wave state:</b> for normal skin and thicker tissue areas.</li> <li><b>Intermittent wave:</b> Inflamed tissue areas or sensitive skin (redness, etc.), Finer skin areas</li> <li>The treatment for the entire face will take from 5-15 minutes. Only eyes: 8-10 minutes.</li> <li>The treatment for the body will range between 10 and 20 min.</li> <li>The course length will vary according to different skin conditions.</li> <li>Working frequency: 1 Mhz</li> </ul>	Never use on dry skin. A conductive gel must be used.	Facial probe Orbicular probe (little corners of face and body) And body probe (when the device is F-801C and F337)
<b>GALVANIC</b>  F-313A F-805 F-803 F-815 F-330 F-332	<ul style="list-style-type: none"> <li><b>Positive pole:</b> to close follicles/ pores after facial treatment</li> <li>Decrease redness, as in mild acne.</li> <li>Prevent inflammation after <i>comedone</i> and blemish treatment.</li> <li>Force acid PH solutions, as astringent into the skin</li> <li><b>Negative pole:</b> to stimulate the circulation of blood to dry skin.</li> <li>To force desincrustation lotion (alkaline PH solution) into the skin</li> </ul>	<ul style="list-style-type: none"> <li><b>Desincrustation:</b> the process to soften and liquefy grease deposits, which are accumulation of sebum in the follicles.</li> <li><b>Ionisation:</b> is the forcing of a solution or cream into the skin.</li> <li>Treatment sessions: maximum 3 times a week. Session of 10-15 minutes.</li> </ul>	Use a <b>cream with polarity</b> and select the electrode polarity accordingly.	<b>Version 1:</b> 2 rollers and an earth straight pole which must be held by the patient.  <b>Version 2:</b> 2 rollers and 2 ball type electrodes. Both electrodes must be used at the same time.

	TREATMENT BENEFITS	REMARKS	PRODUCT	ACCESSORIES
<b>HIGH FREQUENCY</b>  F-314A F-803 F-806 F-812 F-815 F-330 F-331 P-02	<ul style="list-style-type: none"> <li>Accelerate blood circulation.</li> <li>Stimulate glandular activity.</li> <li>Helping the skin excretion and absorption.</li> <li>Promote increased metabolism.</li> <li>Kill dermal bacteria.</li> <li>Generate heat energy within the dermal tissue.</li> <li>Skin cleansing.</li> <li>Prepare the skin prior to galvanic treatment. It improves product penetration into the skin.</li> </ul>	<ul style="list-style-type: none"> <li><b>Direct treatment:</b> The electrode is in direct contact on the patient's skin.</li> <li><b>Indirect treatment:</b> the patient holds the straight electrode and the massage therapist carries out a facial massage or treatment. The beautician will feel a mild current during the massage.</li> <li><b>Colours of electrodes and gas</b> Purple (argon gas) and orange (neon). The orange ones are for skin toning and the blue ones are for disinfection.</li> </ul>	Not necessary	<b>Mushroom shaped electrode:</b> for large areas of skin (face, forehead, neck, etc.) <b>Spade shaped electrode:</b> for middle areas (chin, etc.) <b>Ball shaped electrode:</b> for small areas like nose, etc. <b>Straight electrode pipe:</b> for indirect treatment <b>Comb shaped electrode:</b> capillary stimulation <b>Fork-shaped:</b> neck <b>Roller:</b> big surfaces
<b>VACCUUM AND SPRAY</b>  F-316A F-815 F-331	<ul style="list-style-type: none"> <li><b>Vacuum:</b> Using the desired glass vacuum attachment to aid with the extraction of black-head (<i>comedone</i>), white-head (<i>milia</i>) and acne or blockages of the pores, everyday dirt and grease including stubborn make-up.</li> <li><b>Spray:</b> Use the spray bottle to spray toner or astringent lotion to diminish any visible areas of inflammation or redness. The spray function also helps to calm and comfort the skin.</li> </ul>	<ul style="list-style-type: none"> <li>Use the forefinger over the vacuum release hole on each vacuum cap</li> </ul>	Just spray toner for the spray function	2 spray bottles 3 têtes et 1 applicateur

	TREATMENT BENEFITS	REMARKS	PRODUCT	ACCESSORIES
<b>ROTARY BRUSHES</b>  F-315A F-830 F-331	<ul style="list-style-type: none"> <li>The brush peeling is a mechanical exfoliating process which removes the outer dead cells and dirt of the skin.</li> <li>The speed and brush head should be chosen according to the skin type, skin sensitivity, client comfort and product being used (Eg. A coarse grain exfoliate would require a lower speed than a smooth product type).</li> </ul>	<ul style="list-style-type: none"> <li>Just cleanse the skin prior to brush peeling.</li> <li>Use a steamer, hot towel or hot hammer to open the pores and soften the face area prior to brush peeling.</li> <li>After the peeling, cleanse and tone.</li> </ul>	Exfoliating product.	<b>Brush heads:</b> 5 different brush heads.  3 brushes 2 abrasive heads
<b>EXFOLIATING SPATULE</b>  F-311A F-808 F-330 F-332 F-334 F-335 F-335A P-03	<ul style="list-style-type: none"> <li>Exfoliates the outer dermal layer for deep cleansing through a spatula. Useful for the treatment of Acne and the removal of dead cells, dirt and everyday pollution of skin.</li> </ul> <p>3 types of waves. Treatment should be started with the pulsed wave and apply the other types when the patient feels comfortable.</p> <p><b>Continuous:</b> produces heat on the tissue. Do not apply it for long on the skin to avoid burning the patient. Do not work it at high power. </p> <p><b>Pulsed:</b> It does not heat the skin and for this reason it can be worked at higher power levels. </p> <p><b>Combined:</b> it combines the two other waves. </p>	<ul style="list-style-type: none"> <li>All probes make galvanic function except for F-311A and P-03 (just exfoliation). Just turn the spatula to the other side for galvanic use (F-330, 332, 334, 335, 335A) or choose the right programme on the device (F-808).</li> <li>F-335A: a second probe is provided for smaller areas.</li> </ul>	Exfoliating product for peeling.  Polarised product when using galvanic function	Exfoliating spatula and bracelet.


	TREATMENT	REMARKS	PRODUCT	ACCESSORIES
<b>DIAMOND DERMABRASION</b>  F-834 F-319A F-332 F-334 F-336 F-336B	<ul style="list-style-type: none"> <li>The microdermabrasion with diamond heads uses a mechanical exfoliating system to remove the dead cells on the outer layer of the skin.</li> <li>It is a gentle and safe non-invasive treatment that results in a smooth, fresh and cleaner skin.</li> <li>Used for fine lines and wrinkles, pigmentation or age spots, acne, whiteheads, blackheads, congestion, open pores, scar tissues, etc.</li> </ul>	<ul style="list-style-type: none"> <li>Do not use the device under continuous mode during more than 20 minutes. Wait 10 minutes before starting the machine again to avoid overheating. Do not apply on the same area during too much time in order to avoid burns.</li> <li>Whilst gently exfoliating the surface skin the machine will vacuum the particles of dirt and dead skin away from the target area and contain the debris within a removable filter.</li> <li>Complete beauty treatment:: microdermabrasion + cold hammer + skin scrubber +</li> </ul>	Not used.  After the treatment a product to calm down the skin should be used (cream, cold steam or cold hammer)	Diamond heads of fine aluminium oxide crystal (9 different heads. Different grain):  <b>Ø8 and 9mm:</b> Eyes, nose. Wrinkles, black spots, acne, etc. <b>Ø15mm:</b> Scars, fine lines and skin imperfection. Soft polishing. <b>Ø19mm:</b> Face, big spots, pores, premature ageing for sun exposure, etc. <b>Ø22 mm:</b> Body, specially for acne, elbows, knees and areas with marks.
<b>MICRO-CRYSTAL DERMABRASION</b>  F-336 F-336A	<ul style="list-style-type: none"> <li>The microdermabrasion technique works with microcrystals, which are hard enough to disrupt unwanted tired skin cells. The technique involves propelling the microcrystals across the skin's surface, removing only the top layer of skin cells. Suitable for treating a wide range of skin damage, from age-related wrinkles to dry skin, to sun-related damage and even to acne scars.</li> </ul>	<ul style="list-style-type: none"> <li>Do not apply on the same area under continuous mode in order to avoid burns.</li> <li>Put the device at a higher level than the working one. The applicator must be in contact with the skin making little passes and then, after 3 / 5 seconds, separate it again from the skin.</li> </ul>	Microcrystal. Corindon, 120 micron.  <b>ALV-001:</b> 1 bottle of 2 kg	1 applicator with suction pump to suck away dirt, dead cells and used crystals.  Inlet deposit for the microcrystal.  Outlet deposit for the microcrystal used and waste.

	TREATMENT BENEFITS	REMARKS	PRODUCT	ACCESSORIES
<b>PRESSO-THERAPY</b>  F-826	<ul style="list-style-type: none"> <li>The pressotherapy is a computer controlled compression system, which operates using inflation pumps. The device is equipped with the control unit and the suit (arms, legs, abdomen).</li> <li>It increases blood circulation and lymph flow.</li> <li>Cellulite prevention.</li> <li>It enhances extra-cellular fluid clearance, reduces bloating, swelling and oedema, alleviates leg fatigue and improves oxygen flow through the whole body.</li> </ul>	<ul style="list-style-type: none"> <li>3 programmes and 3 sections to be treated (arms and legs, abdomen and whole body). The sections to be treated are chosen by the SELECT button.</li> </ul> <p><b>PROGRAMS</b></p> <ul style="list-style-type: none"> <li><b>Mode A:</b> Lymphatic drainage.</li> <li><b>Mode B:</b> Liquid retention and cellulite.</li> <li><b>Mode C:</b> Combined.</li> </ul> <p>Power output selection according to patient discomfort.</p>	Always use a disposable paper garment underneath for a hygienic treatment.	1 suit (Arms, legs and abdomen)  Set of pipes

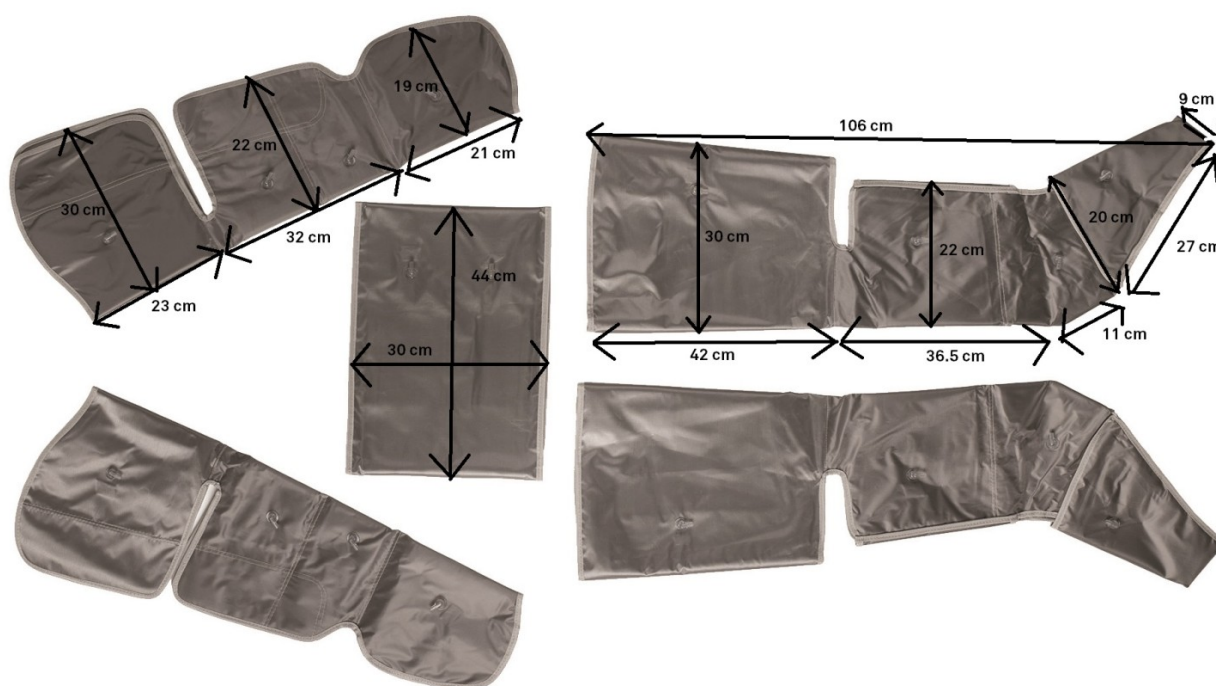



	INFLATION PROGRAMMES AND SEQUENCES
<b>MODE A</b>	<p>The programme is thought for giving a lymphatic drainage massage. It is used for heavy legs and relaxing massages. The inflation process is gradually and slowly performed.</p> <p><b>Inflation sequence:</b> After having chosen the area to be treated by the SELECT button, the different sections will start inflating from the ends. The parts marked with an A will start working, after that the parts B, C and D will gradually and slowly work. Every section is filled with air in 8 seconds. After that, the air is expelled and circulates twice. Afterwards, every section is filled with air in 8-second continuous sequences at the same time.</p>
<b>MODE B</b>	<p>This programme is used for treating the liquid retention and for preventing the cellulite. The sections are inflated quicker than the mode A.</p> <p><b>Inflation sequence:</b> After having chosen the wanted area by the SELECT button, the different sections will start inflating from the ends. The parts marked with an A will start working, after that the parts B, C and D will work quick.</p>
<b>MODE C</b>	<p>Combined programme.</p> <p><b>Inflation sequence:</b> alternating inflation</p> <ol style="list-style-type: none"> <li>1.– Firstly, it starts the section A compression (feet and hands).</li> <li>2.– Section C inflation (forearm and calf)</li> <li>3.– Lower part of the abdomen inflation (E).</li> <li>4.– Section B inflation (wrist and ankle)</li> <li>5.– Section D inflation (femur and biceps)</li> <li>3.– The cycle finishes with the inflation of the upper part of the abdomen (E).</li> </ol>

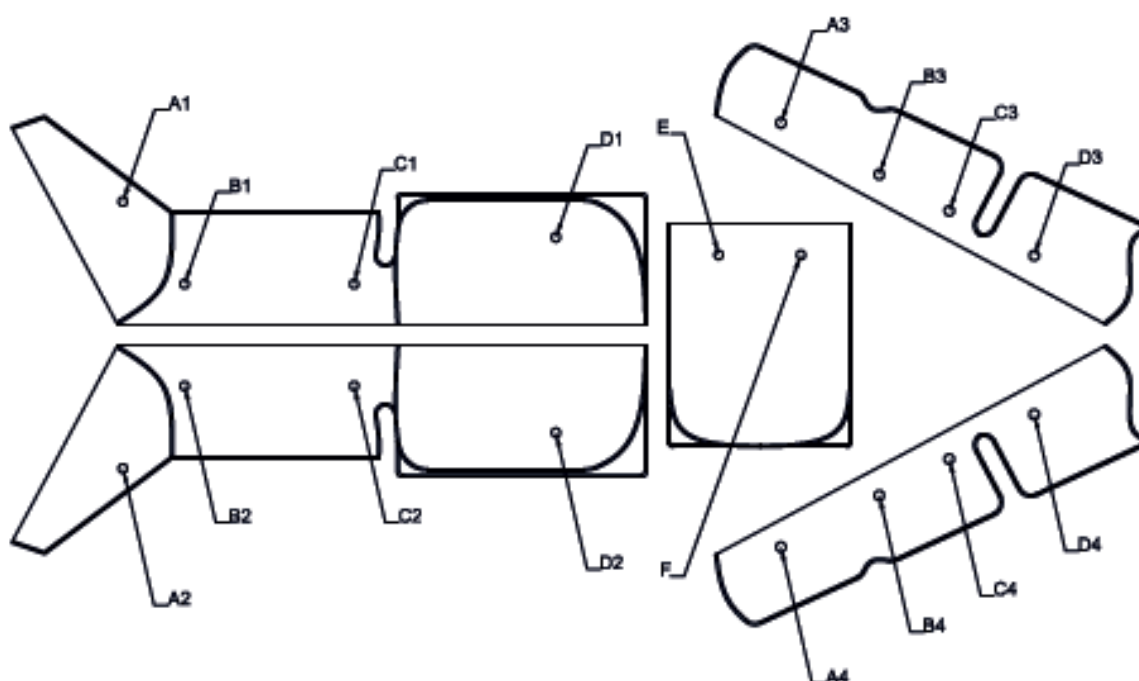






	TREATMENT BENEFITS	REMARKS	PRODUCT	ACCESSORIES
<b>THER-MOTHER APY (Sauna bag)</b>  F-825	<ul style="list-style-type: none"> <li>Thermotherapy works by increasing the temperature of the skin. The Increase of temperature causes vasodilatation, or an increase in the size of the blood vessels, which allows a better blood flow, bringing essential oxygen and nutrients to the tissues faster. Heat is also useful also for soothing muscles.</li> <li>Liquid and toxin elimination</li> </ul>	<ul style="list-style-type: none"> <li>3 different independent heating areas.</li> <li>The adjustment of temperature ranges from 20 to 50° in each section independently.</li> <li>2/3 sessions per week: 40-60 minutes per session</li> </ul>	Use a disposable plastic coating garment underneath for an hygienic treatment and to avoid direct contact with the sauna bag.	Sauna bag



<b>SPOT REMOVAL</b>  F-829	<ul style="list-style-type: none"> <li>It is used for eliminating sun or pigment spots.</li> </ul>	<ul style="list-style-type: none"> <li>Take the applicator as a pen. Do not keep the applicator in contact with skin for more than 1 second. Work gradually on the area to be treated until you see the effects.</li> </ul>	<ul style="list-style-type: none"> <li>Apply a proper repairing lotion to ensure skin renewing after having removed the spot.</li> </ul>	<ul style="list-style-type: none"> <li>Applicator x 1 (10039)</li> <li>Set of 10 single-use needles (40022)</li> <li>Metallic head x 1</li> <li>Longs nails x 4 (40023)</li> </ul>
----------------------------------	--	---	--	--



	TREATMENT	REMARKS	PRODUCT	ACCESSORIES
<b>BODY MASSAGER</b>  F-833	<ul style="list-style-type: none"> <li>Mechanical massage system that transmits vibrating waves to deeper tissues.</li> <li>Reducing the appearance of cellulite</li> <li>Slimming and toning</li> <li>Relaxation</li> <li>Exfoliation</li> <li><b>Applicator 1:</b> for vigorous massage and surface stimulation over large body areas.</li> <li><b>Applicator 2:</b> for vigorous massage and surface stimulation over large body areas and the scalp.</li> <li><b>Applicator 3:</b> Sponge applicator for sensitive areas. Gentle massage.</li> <li><b>Applicator 4:</b> for deep, heavy-pressure massage of major muscle groups such as back, thighs, shoulders and buttocks.</li> <li><b>Applicator 5:</b> is designed for use on the neck, shoulders, legs, arms, back, and buttocks. (10)</li> </ul>	<p>Standard treatment between 30-45 minutes Intensity is adjustable.</p> <p><b>4 stages:</b></p> <ul style="list-style-type: none"> <li>Stimulate lymphatic system and blood flow</li> <li>Exfoliate the areas to be treated</li> <li>Break down the fatty cellulite cells</li> <li>Lymphatic drainage</li> </ul>		<p><b>Applicator 1</b></p>  <p><b>Applicator 2</b></p>  <p><b>Applicator 3</b></p>  <p><b>Applicator 4</b></p>  <p><b>Applicator 5</b></p> 
<b>HOT &amp; COLD HAMMER</b>  F-318A F-331 F-332 P-04	<ul style="list-style-type: none"> <li><b>Hot side of hammer:</b> Muscle relaxation Improve blood circulation Open pores to improve product penetration into the skin</li> <li><b>Cold side of hammer:</b> Reduces wrinkles Calms down skin after microdermabrasion. Close pores / follicles</li> </ul>	<p>Very suitable for physiotherapists in order to apply heat/cold in specific areas.</p> <p>Adjustable temperature range: from 5 to 40°, except for P04, which is not adjustable.</p>		<p>1 double-ended hammer (hot and cold)</p>

	TREATMENT BENEFITS	REMARKS	PRODUCT	ACCESSORIES
<b>RADIO FREQUENCY</b>  F-333	<ul style="list-style-type: none"> <li>• Radio-frequency for face lifting.</li> <li>• Face and neck lifting and remodelling</li> <li>• Tightening and lifting flaccid areas of the face.</li> <li>• Reduces wrinkles</li> <li>• Lift sagging skin.</li> <li>• Enhances luminosity of the skin .</li> <li>• Activates collagen for skin regeneration.</li> <li>• Body sculpture.</li> <li>• Complementary with pressotherapy and cavitation treatments. Used to recover the body tone.</li> </ul>	<ul style="list-style-type: none"> <li>• <b>MONOPOLAR:</b> the electric current emission is performed through an only electrode.</li> <li>• Maximum working power: 150 Watts</li> <li>• Frequency: 0,3 Mhz</li> <li>• Type of waves: <ul style="list-style-type: none"> <li>• Continuous (higher heat effect)</li> <li>• Pulsed (for sensitive skins)</li> </ul> </li> <li>• Treatment time: 10 minutes maximum for facial treatments and up to 30 minutes for body treatment.</li> <li>• 15 days between each treatment.</li> <li>• Risk of burn if not used correctly.</li> </ul>	Cream or conductive gel to be applied	Facial handle with 4 different facial ceramic electrodes of different sizes.  Body handle + 3 body electrodes  Metallic plate for return circuit in both body and face treatments.
<b>HIGH CONDUCTIVITY GLOVES &amp; BIO-SKIN SMOOTHER</b>  F-317A F-330	<ul style="list-style-type: none"> <li>• Electrotherapy with magic gloves:</li> <li>• Facial lymphatic drainage and body drainage</li> <li>• Cellulite treatment</li> <li>• Massage against muscle contracture.</li> <li>• Muscle toning</li> <li>• Facial micro-lifting</li> </ul>	<ul style="list-style-type: none"> <li>• <b>Magic gloves:</b> the beautician uses disposable isolating gloves underneath the magic gloves. Then, the beautician carries out the massage. A micro-current and mild heat is felt by the patient</li> <li>• <b>BIO LIFT:</b> To recover tone and skin elasticity in the face.</li> <li>• Intensive treatment: 30 minute session, once a week.</li> <li>• Preventive maintenance: 30 minute session, once per month.</li> </ul>	-Conductive gel	1 pair of magic gloves  Bio-double fork: 1 pair. These are used for the area around the eyes. Cotton ear plugs must be used for the ears.  Bio-single fork: 1 pair. These are used for the body

	TREATMENT BENEFITS	PROGRAMMES	PRODUCT	ACCESORIES
<b>ELECTRICAL STIMULATION</b>  F-905	<ul style="list-style-type: none"> <li>• <b>Principle of operation:</b> it reproduces the process that occurs when our brain orders muscles to contract. In the case of electrical stimulation, excitation occurs directly on the motor nerves using electrical pulses.</li> <li>• <b>Benefits:</b> <ul style="list-style-type: none"> <li>-Optimising muscular quality.</li> <li>-Lymphatic stimulation.</li> <li>- Toxin elimination.</li> <li>- Body sculpture.</li> </ul> </li> </ul>	<ul style="list-style-type: none"> <li>• <b>Programmes 1 &amp; 5:</b> muscle massage; relieve pain.</li> <li>• <b>Programmes 2 &amp; 6:</b> improves muscular tone in breast, waist, abdomen, buttocks, tights and legs.</li> <li>• <b>Programmes 3 &amp; 7:</b> improves blood system flow and lymphatic drainage.</li> <li>• <b>Programmes 4 &amp; 8:</b> weight-loss programmes.</li> </ul>	Use a neuter hydrogel to apply on the electrode before placing it onto the skin.	10 pairs of small oval electrodes .  10 pair of big oval electrodes.  1 big and 1 small pairs of electrodes for breast.  1 pair of self-adhesive squared electrodes.  10 cables for electrodes.  13 belts to fasten the electrodes.  1 power supply

Fig.1

Fig.2

Fig.3

Fig.4



- **Figure 1:** Big electrode (160 mm) and small electrode (115 mm) for breast.
- **Figure 2:** Small-sized electrodes (65 mm). Since the current density passing through the patient varies according to the area of the electrode and this area needs to adapt as well as possible to the muscle group to be treated, it is advisable to use these electrodes for small muscle areas: arms, calves, etc.
- **Figure 3:** Big-sized electrodes (92 mm). They are recommended for bigger muscle areas like back, abdomen, tights, etc.
- **Figure 4:** Self-adhesive squared electrodes. Dimensions: 47 mm. It is recommended for the shoulders area, one electrode in each side.
- **Belts S:** 4 units. Length: 265 mm; width: 50 mm
- **Belts M:** 4 units. Length: 340 mm; width: 50 mm
- **Belts L:** 2 units. Length: 470 mm; width: 50 mm
- **Belts XL:** 2 units. Length: 880 mm ; width: 50 mm
- **Belt XXL:** 1 unit. Length: 1115 mm; width: 80 mm



**Warning:**

- Place the electrodes correctly according to its polarity (+o-). Each stimulation cable has 2 poles (positive is red connection and negative is black connection)
- Place the electrodes symmetrically by following the instructions given in the manual. The minimum distance between electrodes must be 2 cm.
- The intensity must be set at 0 before starting up the device. Set all knobs at 0.
- P-1 to 4: the output current is strong: Intended for non-sensitive areas.
- P-5 to 8: the output current is weak: Intended for sensitive areas.
- Once the treatment is finished, clean the electrodes with a wet cloth. Make sure that there is no lotion or gel remaining in order not to reduce its lifespan.
- See manual for contraindications.

## FUNCTIONS, TIMES AND PROGRAMMES

INTENDED FOR	TIME	PROGRAMME	POSITION
<b>A. Whole body:</b> It stimulates and improves lymphatic circulation.	25 min.	P3	Reclined, head 30°
<b>B. Lower extremities (legs)</b> Lymphatic stimulation (poor circulation and swelling of ganglion). The electrodes are placed combining all the parts of the legs. (See drawing for placement of electrodes)	30 min.	P3	Lying face down
<b>C. Cellular tissue and lower part of abdomen</b> Toning and strengthening of this muscle group.	30 min.	P3 + P1: muscle relief and increase of its strength	Lying face up
<b>D. Lower extremities (legs)</b> It improves muscle tissue resistance and it drains toxins. The electrodes are placed combining the upper internal part of the leg and the lower internal part. (See drawing of placement)	30 min.	P3 + P1: muscle relief and increase of its strength	Lying face up
<b>E. Thoracic drainage</b> A drainage becomes in the thoracic muscular group and is completed in the abdominal group improving the circulation and toxins and liquids elimination (it is recommended before and after menstruation).	15 min.	P3	Reclined, head 30°
<b>F. Breast / chest toning</b> It contributes to eliminate fatty cells (it is recommended before and after menstruation)	15 min.	P2	Reclined, head 30°
<b>G. Breast / chest firming</b> Loss of elasticity and exercising of the muscle tissue.	15 min.	P2	Reclined, head 30°
<b>H. Treatment of back and abdomen</b> Localised flaccidity. Moulding abdomen and waist. Combined with P3 (lymphatic drainage).	50 min.	P3: 25 min. + P4: 15 min. + P1: 10 min. for muscular massage.	Reclined, head 30°
<b>I. Hips</b> It works to reduce fatty tissues and it moulds hips. Combined with P3 (lymphatic drainage)	50 min.	P3: 25 min. + P4: 15 min. + P1: 10 min. for muscular massage.	Horizontally lying
<b>J. Tights</b> It reduces fat on the front and rear parts of tight and tones (depending on the electrodes placement).	50 min.	P3: 25 min. + P4: 15 min. + P1: 10 min. for muscular massage.	Reclined, raised head 30°
<b>K. Calves</b> It reduces the fat accumulation in calves and strengthen muscles (depending on electrodes placement).	20 min. twice.	P7 o P8. Pay special attention on the current strength. It is a very sensitive area.	Reclined, raised head 30°
<b>L. Arms</b> It reduces fat gathered on the upper and lower part of the arm and strengthen these muscles. It also works the laxity that appears on the lower part of the arm. It is recommended during the menopause (see electrodes placement).	25 min	P3 : 10 min. + P4: 15 min.	Reclined, raised head 30°
<b>M. Arm and shoulder toning</b> Strengthen the cervical and dorsal muscular group and arms. It helps to reduce the excess pressure on the area.	25 min	P3 : 10 min. + P4: 15 min.	Reclined, raised head 30°
<b>N. Back toning</b> It reduces excess fat and strengthen the dorsal and lumbar muscles. It helps to adopt the proper back posture (see electrodes placement)	30 min	P3 : 10 min. + P4: 15 min.	Standing



*This information is complementary to the instruction manual of the device. See the graphic placement of electrodes in the user guide.*

	WEELKO STEAMERS
USE	<ul style="list-style-type: none"> <li>• Soften the skin and the dead cells before applying facial cleansing and later treatments.</li> <li>• Open pores.</li> <li>• The steam helps to release natural toxin of the skin. It also contributes to soften the wrinkles before the treatments.</li> <li>• Increases the blood flow, hydrates the skin and improves the metabolism and the skin regeneration.</li> <li>• Helps in the acne treatments.</li> <li>• If used with herbs, it helps to nourish and light the skin.</li> <li>• Portable facial steamer.</li> </ul>
CAUTIONS AND WARNINGS	<ul style="list-style-type: none"> <li>• Use only <b>distilled water</b>.</li> <li>• Keep the reservoir full of water between the minimum and maximum line.</li> <li>• Do not use the steamer without water in the reservoir.</li> <li>• Check if the watertightness joint and the reservoir are well-placed.</li> <li>• Do not tighten the reservoir too much as this could damage the joint or the reservoir could get cracked while the boiling process.</li> <li>• When the water reaches the lower limit, the steamer gets automatically stopped.</li> <li>• Pay special attention to not polluting the water with the fragrances or herbs because the steamer could expel drops of water altogether with the steam and this might scald the patient.</li> <li>• The water reservoir reaches very high temperatures. Do not touch it with your hands.</li> <li>• Do not point directly at the patient's face. Once the steam is uniformly sprayed out, the steamer should be slowly moved from the shoulders to the face. (Keep a minimum distance of 45 cm from the patient's face).</li> <li>• Do not use in wet places.</li> <li>• Avoid sun exposure.</li> <li>• Do not manipulate the steamer inside.</li> <li>• <b>F-300DT:</b> To remove the water reservoir, unscrew firstly the reservoir base from the main structure.</li> </ul> <div style="display: flex; justify-content: space-around; align-items: center;">   </div>
SUGGESTIONS FOR THE TREATMENT	<ul style="list-style-type: none"> <li>• <b>Greasy skin and bad blood circulation:</b> 35-40 cm distance. Time: 15 minutes approximately.</li> <li>• <b>Normal skin:</b> 35-40 cm distance. Time: 10 minutes approximately.</li> <li>• <b>Sensitive and dry skin:</b> 40-45 cm distance. Time: 5 minutes approximately.</li> <li>• <b>Expanded capillary and inflammation:</b> 40 cm distance. Time: 5 minutes approximately.</li> </ul>



MODEL	OZONE	HERBS	FRA-GRANCES	TIMER	NOTES	OPTIONAL
<b>PORT</b> F-100C					PORTABLE	
<b>ATHMOS</b> B-002					5-WHEEL BASE	
<b>PURE</b> A30					5-WHEEL BASE	
<b>CLEAR</b> F-800A				YES. MANUAL	5-WHEEL BASE	Lamp support (40011) Lamp1001T
<b>DUO</b> FD-2103					TWO RESER-VOIRS COLD / HOT	
<b>CROMO</b> F-300H					DIGITAL	
<b>H-TWO</b> F-300A				15 minutes	DIGITAL. Steam adjustment	Lamp support (40011) Lamp1001T
F-300DT				YES. DIGITAL	For towers 3021 / 3051 / F-300E Steamer adjust-ment	
<b>VAP</b> F-300E				YES. DIGITAL	On a mobile trolley with a rotary brushes equipment	Lamp 1005T



## WHEELKO STERILISERS

	Ultrasure UC-001	Termosept S-01
<b>MODEL</b>		
<b>DESCRIPTION</b>	Ultrasound steriliser with a plastic cover and a digital timer. Capacity: 1,3 l	Quartz beads steriliser with a plastic cover and a digital timer. Capacity: 152 ml.
<b>PROCESS</b>	Equipment to clean and disinfect by the descaling process that comes out of its ultrasonic waves.	Equipment to clean and disinfect objects in a quick process at a very high temperature in a reservoir of quartz beads. The unit heats up previously to the sterilisation process from 160 to 180°. Once it reaches the minimum temperature, introduce the tools to be treated until the process is finished. The beads protect the objects from the heat and eliminate the dirt and the bacteria.
<b>TIME OF STERILISATION</b>	Estimated time for the sterilisation process: 3 minutes.	Estimated time for the sterilisation process: 20 seconds.  Time to reach the maximum temperature previously to the treatment: 15 minutes.
<b>OBJECTS TO BE TREATED</b>	Medium-sized objects made of plastic or metal, such as combs, razor blades, tweezers or forceps, etc.	Small-sized metallic instruments.
<b>NOTES</b>	To carry out the steriliser bath, introduce clean and cool water into the reservoir without surpassing the maximum capacity. Do not use alcohol.  If an intensive cleaning is required for very dirty tools, you can add some drops of a liquid detergent or a body soap.	The quartz beads last between 3 and 4 months. To extend their useful life, the beads can be cleaned by a cotton cloth after each process.  Once the cycle is finished, it is recommended to take the items out by some tweezers in order to avoid getting burnt, because the reached temperature is very high during the process.

## WEEKKO STERILISERS

	UV-Power S-02	Warmex T-01
<b>MODEL</b>		
<b>DESCRIPTION</b>	Ultraviolet steriliser with an acrylic drawer. Capacity: 3,5 l.	Towel heater and ultraviolet steriliser with a glass door that allows seeing the inside. There is a removable drip tray included to remove the spare water once the process is finished. Capacity: 7 l.
<b>PROCESS</b>	Germicidal equipment that cleans, disinfects and eliminates the bad smells by its ultraviolet lamp.	Ultraviolet steriliser and small towel heater. Ideal for reaching a comfortable sensation in facial treatments and to eliminate the germs on the towels. It can also be used for sterilising other instruments. A switch activates the heat function and another one regulates the sterilisation function. Both functions can work at the same time.
<b>TIME OF STERILISATION</b>	Estimated time for the sterilisation process: 30-45 minutes.	Estimated time for the sterilisation process: 30 minutes.
<b>OBJECTS TO BE TREATED</b>	Medium-sized plastic instruments, specially non-cutting and blunt ones. Ideal for combs, brushes and scissors.	Small towels and medium-sized instruments, such as brushes, combs, etc.
<b>NOTES</b>		<ul style="list-style-type: none"> <li>• The maximum temperature is 60°.</li> <li>• Do not introduce soaked towels. The equipment aim is heating, but not drying.</li> <li>• It can be combined with the multifunctional tower.</li> </ul>

Slimming Pad (F-905)  
Ref. 10001L



Slimming Pad (F-905)  
Ref. 10001S



Slimming Pad (F-905)  
Ref. 10002L



Slimming Pad (F-905)  
Ref. 10002S



Slimming Pad (F-905)  
Ref. 10003L



Galvanic Roll And Stick  
Combined Series  
Ref. 10004



High Frequency Electrode 1  
Ref. 10009



High Frequency Electrode 2  
Ref. 10010



High Frequency Electrode 3  
Ref. 10011



High Frequency Electrode 4  
Ref. 10012



Skin Scrubber Probe  
Ref. 10018



Belt For Skin Scrubber  
Ref. 10019



High Frequency Electrode 1  
(Special)  
Ref. 10024



High Frequency Electrode 2  
(Special)  
Ref. 10025



High Frequency Electrode 3  
(Special)  
Ref. 10026



Vaccum Glass Set 3p (F-316&F-807A)  
Ref. 10027



Vacuum Handle (F-316 & F-807A)  
Ref. 10028



Vacuum Tubes (F-316 & F-807A)  
Ref. 10029



Spray Bottle (Set Of Two)  
Ref. 10033



Ultrasonic Eyes Probe  
Ref. 10034



Ultrasonic Face Probe  
Ref. 10035



Ultrasonic Body Probe  
Ref. 10036



Ultrasonic Eyes Probe Small  
Comb. Series  
Ref. 10037



Ultrasonic Face Probe Small  
Comb. Series  
Ref. 10038



Applicator Spot Removal  
Ref. 10039



High Frequency Handle  
Ref. 10040



Galvanic Rolls Set  
Ref. 10042



Galvanic Double Stick Set  
Ref. 10043



Belts Set For F-905  
Ref. 10045



Water Deposit (F-800A)  
Ref. 10048



Water Deposit (F-100C)  
Ref. 10051



Water Deposit (B-002)  
Ref. 10052



Water Deposit (A30, FD-2103)  
Ref. 10054



Water Deposit (F-300A)  
Ref. 10060



Clamp For Lamp  
Ref. 10062



Fluorescent For 1001  
Ref. 10070



Infrared Bulb  
Ref. 10072



Brushes Set  
Ref. 10073



Brush Gun For F-315A (Black)  
Ref. 10074



Brush Gun For F-830 (White)  
Ref. 10074-8



Handle For Bio-Lifting  
Ref. 10097




Handle For Bio-Lifting  
Ref. 10098




Ultrasonic Body Probe  
Ref. 10103



Ultrasonic Face Probe  
Ref. 10104

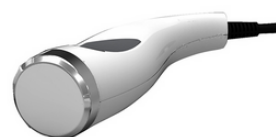

Ultrasonic Eyes Probe  
Ref. 10105

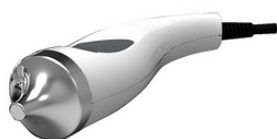

Air Pressure Suit  
Ref. 40001



Right Boot For F-826  
Ref. 40001-1



Left Boot For F-826  
Ref. 40001-2



Right Arm For F-826  
Ref. 40001-3



Left Arm For F-826  
Ref. 40001-4



Waist For F-826  
Ref. 40001-5



Tubes For F-826  
Ref. 40001-6



Sauna Bag  
Ref. 40002



Glasses Bag  
Ref. 40003



Diamond Heads Set  
Ref. 40010



Foam&Metallic Filter For  
Microdermabrasion  
Ref. 40010-1



Support For Lamp  
Ref. 40011



Dermabrasion Handle  
Ref. 40016



Magic Gloves (Set Of Two)  
Ref. 60038



Filter For Micro Cristal  
Ref. 40018



Rubber Bag For 1031  
Ref. 50046



Microcrystal 2Kg Recipient  
Ref. ALV-001



Hand Controller 1 Motor  
Ref. HC-001



Hand Controller 2 Motors  
Ref. HC-002



Hand Controller 3 Motors  
Ref. HC-003



Hand Controller 4 Motors  
Ref. HC-004



Hand Controller 5 Motors  
Ref. HC-005





weelko.com

**Barcelona**

Pol. Ind. La Valldan  
Camí de Garreta, 92  
08600 Berga  
Barcelona - Spain  
T. +34 93 824 90 20  
E. info@weelko.com



Fast



Easy




Good




**weelko**



Safe

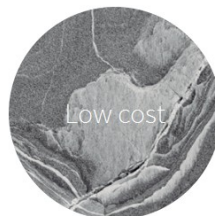

Low cost

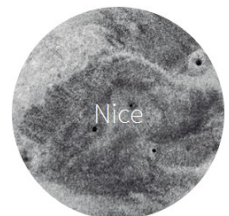

Nice

**IN GOOD SHAPE**



weelko

weelko.com

**Barcelona**

Pol. Ind. La Valldan  
Camí de Garreta, 92  
08600 Berga  
Barcelona - Spain  
T. +34 93 824 90 20  
E. info@weelko.com

**Showroom**

C. Muntaner, 538  
08022 Barcelona - Spain  
T. + 34 93 11 01 030




weelko



Safe



Low cost

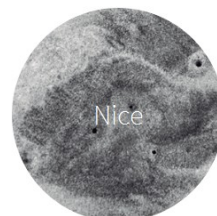

Nice

**IN GOOD SHAPE**



weelko