Microdermoabrasión

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Microdermabrasion

Microdermabrasion equipment that combines two techniques: using diamond tips and aluminium oxide microcrystals. Ideal for controlled, precise and progressive exfoliation to improve skin quality, reduce blemishes, smooth expression lines and treat scars.

Indicated in treatments to eliminate blackheads, acne, accumulated dirt on the skin. Also to revitalise, nourish, increase the elasticity and oxygenation of the skin, by means of.

Microdermabrasion with diamond tip, is a non-surgical procedure that is performed at the level of the corneal layer and seeks the renewal of the epidermis with mechanical exfoliation, helping to remove dead skin by sanding and sweeping, producing cell regeneration, increasing collagen production, and helping to improve the texture and colour of the skin making it more lush and smooth.

Microdermabrasion with microcrystals (aluminium oxide) powder is a very effective, painless and precise mechanical exfoliation method, in which the handpiece spreads the microcrystals on the skin and at the same time causes erosion and a vacuum that drags the dead skin cells and product residues, which are absorbed into the deposit that receives the remains and disposes of them in a very hygienic way. In this case, the impact of these particles against the tissue tends to achieve greater depth in fewer passes.

Both procedures can be performed on the face, neck, décolleté, arms and hands, in order to encourage the formation of new cells to replace those removed. At the end of the microdermabrasion you may feel a burning sensation and dryness of the skin in the area that was exposed to the procedure, this is normal, to reduce the redness it is advisable to use the cool hammer. However, depending on the intensity of the procedure, the skin may become a redder colour which should disappear in the following hours, but if the discomfort continues, it is preferable to consult your doctor.

To activate the device, select Function Microcrystal or diamond dermabrasion and set the suction intensity in energy.

The skin should be clean and dry. Avoid sun exposure for at least 1 week before. Refrain from using products containing retinoids or glycolic acid for 10 days beforehand.

Microdermabrasion

Non-surgical procedure that is performed at the level of the corneal layer of the skin and seeks the renewal of the epidermis through mechanical exfoliation, helping to remove dead skin by sanding and sweeping with a diamond tip, producing cell regeneration, increasing collagen production, and helping to improve the texture and colour of the skin making it smoother and softer.

Diamond tip microdermabrasion is a controlled, non-invasive mechanical exfoliation treatment that removes dead skin cells. It improves the appearance, tone, radiance and restores elasticity. Different diamond tips (different steel crowns) with different diameters and granulations can be placed on the handpiece, to be chosen according to the area to be treated. This slides and rubs over the skin, removing the layers of dead skin and the vacuum generated by the equipment in turn sucks up the dead cells.

Ø 22 MM	
D100	Body. Conceived for acne, knees, elbows and areas with stretch marks.
D180	
Ø 19 MM	
D180	Face. Conceived for big spots, pores, aging effects for sun exposure.
D220	
Ø 15 MM	
D100	Scars, fine expression lines and other imperfections.
D200	
D250	
Ø 8 Y 6 MM	
D200	Eyes and both sides of the nose. Wrinkles, acne, pimples and blackheads.
D250	


Microdermabrasion

How to use:

- Connect the tube to the machine and to the microdermabrasion handpiece.
- Place a foam filter before threading the diamond tip. Remove the filter after each treatment.
- Select the tip you wish to use, and then screw it onto the tip of the handpiece, until the rubber O-ring is tightened.
- Adjust the suction intensity with the vacuum regulator according to the table below, keeping in communication with the client for safety.
- Bring the handpiece with the appropriate diamond tip into contact with the skin. Proceed with a slow and smooth gliding motion. Do not leave the handpiece unmoved for too long on the same skin surface, as this may cause injury.
- Gently stretch the skin with one hand while gliding the diamond tip 2 or 3 times over the same area. Respect this point to avoid redness or injury to the skin.
- After the microdermabrasion treatment, cleanse the skin of impurities and dead skin with a soft brush, damp sponge or damp towel.
- Use a brush or cotton swab lightly moistened with alcohol or a disinfectant to clean and disinfect the diamond tip. It is also recommended to sterilise the tips used after each treatment in an ultraviolet steriliser or at high temperature.

Diamond heads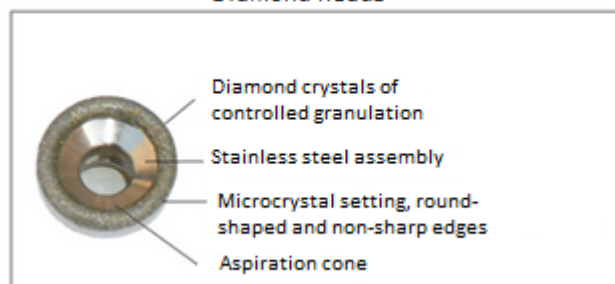

Precautions:

- Avoid use on irritated or injured skin areas.
- When decreased suction is noticed, replace the filter.
- Do not use coarse-grained tips on delicate areas.
- Clean the tips used on each person.
- Avoid bumping and dropping the tips, as this can cause damage and loss of polishing power.
- The microdermabrasion treatment cannot be applied on sensitive areas such as the eyelids, and the eyes must be closed during the treatment, it is advisable to cover them with wet cotton pads.
- It is contraindicated in the following cases: skin tumors, moles, birthmarks, angiomas, eyes, herpes and warts.
- Do not attempt to repair the equipment. In case of failure, contact your distributor.

Microdermoabrasion with microcrystals

Microdermoabrasion with microcrystals (aluminium oxide) powder, on the other hand, is a very effective, painless and precise mechanical exfoliation method, in which the handpiece spreads the microcrystals on the skin and at the same time causes erosion and a vacuum that drags away dead skin cells and product residues, which are absorbed into the deposit that receives the remains and disposes of them in a very hygienic way. In this case, the impact of these particles against the tissue tends to achieve more depth in fewer passes.

Easy to use and noticeably improving the appearance of the skin, this equipment sprays fine crystals quickly and precisely onto the surface of your skin to produce exfoliation, while dead skin is removed by vacuum suction. The skin will be refreshed and improved after the treatment, it will also stimulate circulation and improve metabolism. Upward, gentle, sweeping movements should be made.

Benefits:

- Improves the appearance of aging skin, acne scars, and removes dirt from pores.
- Renews dull, lackluster skin.
- Improves facial oedema and inflammation.
- Improves skin elasticity, fine wrinkles, texture, giving it a youthful and fresh appearance.
- Improve uneven skin tone, pigmentation and other problems.

How to use:

- Install the equipment, make sure the main body and accessories are well connected, as indicated in the manual. The LCD screen lights up and displays "Microcrystal". The equipment will be in standby mode.
- Place the product in the container.
- The default time is 15 minutes.
- Press the "+ / -" buttons on the timer to select a working time range.
- Press the "+ / -" button of "ENERGY", to adjust the suction power.
- Press the "START/PAUSE" button to start working, while the LCD display shows the working status.
- When the preset timer has run out, you will hear a "beep" sound. The timer will reset after 15 minutes.

Microdermoabrasion with microcrystals

Precautions for use:

- Analyse the client's skin type prior to treatment, and select the appropriate strength according to needs.
- Thoroughly cleanse and dry the skin, no product application is required.
- After the treatment, cleanse the facial skin and apply a moisturising or nourishing product.
- The client should avoid exposure to the sun.
- It is recommended not to use make-up 4 hours after the treatment.
- Wash the face with cold water, and avoid exposure to the sauna.
- A certain time interval should be allowed for a new treatment.
- After 2-3 days the client may experience itching and flaking, so skin protection and care is essential.
- After exfoliation, please do not use acid type products.
- The size of the applied crystal should be 180-200mesh. It is disposable and should not be recycled.
- After treatment, use water and a brush to clean the accessories, then immerse them completely in the container for disinfection using 75% alcohol solution for 15 minutes. Afterwards, remove, clean and place them in the UV steriliser for next use.
- It is important to cover the client's eyes during treatment on the face to prevent the crystals from entering the eyes.

F336

Indications

- Combat uneven texture and wrinkles.
- Deep facial cleansing.
- Collagen regeneration.
- Dehydration.
- Improve the signs of ageing.
- Acne.
- Improve skin hydration and eliminate the appearance of fine lines.
- Hyperpigmentation and reduction of spots, sun spots.

Care

- The use of sunscreen is recommended after each session.
- Drink 1.5 to 2 litres of water a day to keep the skin hydrated.
- Maintain daily skin care habits, cleanliness and use of creams suitable for the type of skin.
- Facial skin, being thinner, tends to be very delicate and at greater risk of burns, so at no time should any of the handpieces be left in place, and microdermabrasion should be carried out gently.
- In some cases the treatment may cause reddening of the skin, irritation and swelling, which will improve with time.
- It is important to clean the handpieces properly after each procedure.

Warnings

- This treatment is not risk-free, which is why it must be carried out by a professional aesthetic specialist.
- Follow at all times the recommendations of the professionals of the aesthetic centre.
- Never, under any circumstances, attempt to open or inspect the internal components or accessories of the machine. If inspection is necessary, please contact your supplier or a qualified technician.
- Never use the unit in damp or insufficiently ventilated environments, such as water areas, bathrooms, etc., in order to prevent damage to the electrical components of your equipment.
- Never use the equipment outdoors, inclement weather may cause damage.
- If the electrical cable is broken, ask a professional to supply you with another one and replace it, in order to avoid risks, or contact the supplier.
- At the end of the session, it is important to clean the handpieces with an antiseptic product after each treatment.
- Check with the client throughout the treatment that the intensity is appropriate.
- It is recommended that the client drinks plenty of water before and after the treatment.
- Do not place any liquid on the equipment.
- Be careful with electrical connections.
- Please disassemble and assemble the equipment by professional personnel.
- In case of malfunction of the device, check all connections carefully and contact a service technician in case of doubt.

Contraindications

- Pregnancy and breastfeeding.
- Very sensitive skin.
- Rosacea.
- Seborrheic dermatitis.
- Recent lesions, skin with wounds, infections, psoriasis, dermatitis or unhealthy skin.
- Shaved or depilated skin.
- Skin that has recently undergone deep chemical peeling, burns or surgery.
- Alterations in skin sensitivity, unable to differentiate between cold and heat.
- Poor healing.
- Previous history of keloids.
- Areas with tattoos on the face or permanent make-up.
- Skin conditions.
- Infectious processes or fungus.
- Severe acne.
- Caution when working on tissues with injected fillers, and in the case of clients who have recently had hyaluronic acid, Botox or any other type of product injected, they should wait a reasonable amount of time before having treatment.

Yubitzay Avila
Customer Training
T. +34 93 824 9020
y.avila@weelko.com
S. Yubitzay Avila

Weelko Barcelona S. L.
Pol. Ind. La Valldan
Camí de Garreta, 92
08600 Berga
Barcelona – Spain
T. +34 93 824 9020
info@weelko.com
www.weelko.com