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Body massage equipment, which generates a rotating vibratory movement, resulting in the following effects:

- Increased blood, venous, lymphatic circulation, improving cell metabolism and cellulite.
- Improves the transport of nutrients, as well as oxygenates the cells and muscles.
- Increases the elimination of toxins and liquids.
- Generates multiple benefits to the skin as it improves dry skin, texture by removing dead cells from the surface and stimulates its function by increasing the activity of the sebaceous glands.
- Generates relaxation of tense muscles due to the warming effect.
- Relieves muscle tension, aches and pains due to the rapid elimination of waste products.
- Helps drainage of soft and hard adipose tissue areas.
- Reduce localized fat deposits.

The massage head is activated by a powerful electric motor, which rests on a pedestal, has air outlets that must not be obstructed and has a flexible hose that connects the motor to the vibrating head. This head has interchangeable parts with different applications at body level, which gives the professional greater amplitude in the therapeutic effects of the massage. The intensity of the treatment is controlled by the knob, to work at deep or superficial level.

Mode of use:

Once the client has been evaluated and verified that there are no contraindications to the treatment, we will proceed to start the with the massage.

To maintain the hygiene of the equipment it is recommended to disinfect both the work area and the handpiece before and after the treatment, a cover can be placed on the head to protect it and we will make sure that the equipment is stable.

We take the head and turn on the equipment with caution, moving the head away from the client, always starting with the intensity from less to more, according to the client's tolerance. The surface of each head should be parallel to the surface of the body, adapting to the contour. It should not be used on bony areas or on the abdomen with poor muscle tone.

The maneuvers performed will ensure the effectiveness of the treatment and the safety of the client, so it is very important to perform upward movement in the case of the legs, always in the direction of the natural drainage of the body. The therapist can perform manual massage movements with the mechanical applicator, combining the depth of the vibratory treatment with the personal touch.

The recommended application time per zone should not exceed 30 minutes.

The number of sessions can vary between 5 to 10, 2 to 3 per week. The treatment will depend on the area of the body to be treated, the client's metabolism, and the results we are looking for.

It has 5 heads:

Soft and versatile circular sponge head. For sensitive parts of the body. Light and gentle massage. Used before and after any treatment to prepare the tissues and decontract. Speeds between 3 and 5.

It is recommended to use talcum powder on the area when using this sponge, as the oil can damage it, it is widely used to start and warm up the treatment area with soft movements, effleurage, helping us with the other hand to generate rhythmic movements.

Circular head with large tips for deep massage. Use with low speeds. Between levels 1 and 2. At maximum speed (level 5) it is also used to treat areas of solid subcutaneous fat, activation of circulation, rough skin and anti-cellulite massage. The deeper the pressure on the adipose tissue and the increased circulation in the area will help the dispersion of fat deposits.

Circular head with multiple fine silicone tips that acts at the epidermis level. Stimulation, exfoliation, vasodilation and invigorating massage treatments. Used on the back, buttocks and thighs. Use with speed 5. Prepares the skin to facilitate the penetration of slimming creams, draining creams or body oils. Used in sports treatments before exercise due to the hyperemiant action generated locally with the massage (levels 4 and 5).

Square applicator with 4 hemispheres. Used with deep pressure on fleshy areas such as buttocks and thighs. Speeds 1, 2 and 3.

V-Shaped sponge applicator. This shape is conceived for using it in shoulders, neck, back, cervical area, legs and knees. Suitable for relaxing and anti-stress treatments. Use a fiber or paper cover to protect the applicator during its use. Use it with speeds 2, 3 and 4.

It is recommended to use talcum powder to work with the handpieces, and the applicator heads (not the sponge ones) can be washed with hot water and soap, or cleaned with disinfectant to remove talc, dead skin cells and sebum. They are then dried and can be disinfected in an ultraviolet machine. Disposable protective covers or fiber or paper covers can also be purchased to cover the head during use.

Finally, a massage or manual drainage can be applied to soothe the skin after the treatment. And the treatment can be complemented by offering a different service, skin care, firming and toning. Too strong an application, too much, incorrect choice of head for the treatment area or incorrect and prolonged application can cause damage, erythema, dilation of capillaries and hematomas. The advantage of this equipment is that it produces a much deeper massage. It is much less strenuous for the therapist.

Contraindications

- Inflammation and skin disorders.
- Thrombosis / phlebitis.
- Open wounds, cuts or abrasions.
- Scar tissue.
- Prominent bony areas.
- Vascular disorders.
- Hypersensitive or very thin skin.
- Sunburn.
- Edema, recent fractures.
- Vascular conditions, varicose veins, thrombophlebitis and lymphagitis.
- Excessively hairy areas.
- People with heart problems.
- People wearing pacemakers.
- Infectious and inflammatory diseases in their most acute phase.
- Pregnancy.
- Tumors.
- High and/or low blood pressure problems.
- Cardiovascular problems, phlebitis or thrombosis.
- Diabetes.
- Epilepsy.
- Eczemas.
- Psoriasis.

Yubitzay Avila
Customer Training
T. +34 93 824 9020
y.avila@weelko.com
S. Yubitzay Avila

Weelko Barcelona S.
L.
Pol. Ind. La Valldan
Camí de Garreta, 92
08600 Berga
Barcelona – Spain
T. +34 93 824 9020
info@weelko.com
www.weelko.com