UC001

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UC001

The UC001, an ultrasonic instrument cleaner, makes use of physical ultrasonic vibration at a high speed together with a suitable product to thoroughly clean all debris and impurities adhering to the surface of the objects, but it is not effective without a cleaning solution suitable for the material and the dirt.

Disinfectants may be used, provided they do not react with the materials being cleaned or with the equipment material.

This equipment has a protection function and an alarm system which, when working without water, prevents damage to the equipment. The electrical energy is converted into high-speed mechanical energy that vibrates at a rate of 40,000 times per second, and the water dissolves any impurities, debris or dirt adhered to the surface, producing a sound during the process.

The tray of the UC-001 is made of plastic and the interior of the equipment is of stainless steel, if it remains whitish it can be for the quality of the water that is not totally pure, remaining sediments after working with the equipment, for this we recommend to use distilled water.

The utensils that can be cleaned in this equipment are: stainless steel, aluminium, metal, glass, copper, wood, plastic, rubber and fabric.

It should be noted that this equipment is not for removing or descaling rust from tools, and cannot be used with anti-corrosives.

Because it uses ultrasonic vibration to clean, the water temperature does not increase. But if used for a long time, it may cause slight heating of the metal coating. Generally, it is recommended to use it for 3 minutes at a time to clean the dirt on the surface of utensils.

After cleaning, drain the water and use a towel to wipe the tray and bowl.

How to use:

- Place the material in the tray.
- Add the appropriate amount of water or distilled water as indicated, without exceeding the maximum level.
- Add disinfectant or descaling solution compatible with ultrasonic cleaning.
- Close the lid of the cleaner, connect the electricity, turn on the switch.
- The default working time is "3" minutes, but it can be used several times depending on the cleaning needs of the instruments.
- Press the start-up/pause button to start, if we press the "start-up/pause" button again, the cleaner stops working.
- When it is running, the indicator lamp will be illuminated. In this time, if the top cover of the machine is opened, within 3 seconds the alarm system will start to sound, and the cleaning function will stop, the indicator lamp will go out, and the timer will be displayed in pause status, and you can open the equipment and take out the material.
- After using the equipment, turn off the switch, unplug it, and use a towel to dry the materials.

Precautions

- Follow the instructions.
- Do not use the machine near high temperature objects or appliances.
- Please do not use the machine in the presence of acid or explosive gases.
- Do not move the machine while it is working or connected to the mains.
- When the machine is running it will emit a characteristic buzzing sound.
- When working continuously, the water temperature will normally rise.
- Once the cleaning process is finished, remove the material, switch off unplug the machine.

Maintenance:

- While the machine is in operation, the temperature of the internal parts will rise, do not use the equipment intensively.
- When the cleaning process is finished, proceed to change the water in the tank and dry it properly to avoid the formation of scale.
- Do not place the material directly into the tank without the protective grid.
- Do not use acidic or alkaline substances.
- The voltage is normal to safeguard the safety of the equipment and the user.

Yubitzay Avila
Customer Training
T. +34 93 824 9020
y.avila@weelko.com
S. Yubitzay Avila

Weelko Barcelona S. L.
Pol. Ind. La Valldan
Camí de Garreta, 92
08600 Berga
Barcelona – Spain
T. +34 93 824 9020
info@weelko.com
www.weelko.com