

# Diagnos 1004



## Lámpara Wood

It is an ultraviolet light diagnostic lamp that allows us to examine the skin in depth and detect any problem that is not evident to the naked eye.

For best results, the skin must be clean, without make-up or products, the place should be as dark as possible so that the face can be seen well under the light, the lamp is placed about 10 cm away from the face, you should not look directly at the light, and the professional must differentiate the colours of the skin to diagnose the problem.

Depending on the part of the body where the lamp is applied, there may be variations in the colours depending on the condition of the skin:

On the hair and head it will show white and grey-purple to indicate healthy skin.

On the nails it will show bright white to indicate a healthy state.

### Benefits:

Helps to detect all kinds of problems such as bacterial infections, fungus, pores, oil level, sun damage, flaking, blemishes, photosensitivity, photo-ageing or colour changes.

It helps to diagnose imperfections in all skin types from the face to the hands.

It can also detect lack of hydration, excess acne, skin blemishes, redness, and sometimes even tumours and pigmentation problems on the skin surface.

It is painless, as it is a non-invasive and extremely effective method.

To use the lamp on the face, the following steps must be followed. First, attach the sun protection cap to the lamp, and then attach the other end of the sun protection cover to the client's face. The operation should be done in a dark room.

# Diagnos 1004



Diagnosis according to the colour shown:

**Orange:** acne, mycosis, fungi and presence of bacteria.

**Fuchsia:** rosacea, sensitive skin.

**Pink:** dehydration.

**Blue and white:** normal, healthy and hydrated skin.

**Red or brown:** dark spots.

**Violet:** dehydrated skin.

**Purple:** dehydration in thin skin.

**White (fluorescent):** dehydration, dead cells, keratinised areas, thick skin (epidermis).

**White dots:** cell necrosis.

**Green:** comedones, sebaceous skin.

**Yellow:** oily skin, pustules and acne.

**Bright fluorescent:** abundant hydration.

**Yellow or pink:** oily skin.

## Precautions when using the lamp

- The lamp must never come into contact with the skin.
- While using the lamp, the client should close his eyes or cover them with a cotton cloth. Do not look directly into the light beam of the lamp with open eyes at any time.
- The lamp should not be used for a long period of time. It is advisable for the beautician to monitor the operating time of the lamp.
- Before using the lamp, the client's skin must be clean. There must not be any traces of make-up or beauty products on the skin.



Yubitzay Avila  
Customer Training  
T. +34 93 824 9020  
y.avila@weelko.com  
S. Yubitzay Avila

Weelko Barcelona S. L.  
Pol. Ind. La Valldan  
Camí de Garreta, 92  
08600 Berga  
Barcelona – Spain  
T. +34 93 824 9020  
info@weelko.com  
www.weelko.com