

F316A



Beauty equipment that combines 2 functions: vacuum & Spray. Indicated for cleansing, revitalizing and moisturizing, nourishing treatments and oxygenation of the skin.

Vacuum

The facial vacuum system is a suction method, used to increase blood flow and circulation, facial remodeling and skin stimulation, the continuous and controlled suction activates the different tissues of the face and stimulates microcirculation, producing tissue regeneration through massage. It is ideal to prevent aging, increase elasticity and restore vitality to the skin. The vacuumtherapy has 3 suction heads of different sizes and a manipulator. To activate it, select the vacuum function and adjust the suction on the knob. Vacuum therapy is an excellent complement to facial and body treatments. Use the small glass suction cup for blackheads, blackheads with whiteheads, acne, dirt and grease.

How to use:

Connect the air tube to the corresponding nozzle of the vacuum socket, the other end should be connected to the small glass suction cup (There are four small glass suction cups to choose from depending on the area to work on, or to perform the vacuum blackhead removal operation). Turn on the "POWER" button and adjust the degree of suction/vacuum according to the need. It is advisable (for the cleaning and longevity of the equipment) to insert absorbent cotton in the glass vacuum couplings to act as a disposable filter, preventing dirt from being sucked into the tube and thus avoiding damage. The suction cup should not be held against the skin for too long to avoid bruising. Please note that it is normal that the vacuum pressure is not instantaneous at the beginning, it will take some time before the vacuum.

F316A



Facial Spray

Use the spray to reduce inflammation and to soothe and nourish the skin. Spray with toning, soothing or astringent lotion, the spray allows us to spray active ingredients on the skin, for a better diffusion and effectiveness of the treatment.

Connect one end of the air tube to the corresponding "spray" socket, the other end should be connected to the small spray bottle. Two spray bottles are supplied and can be filled with different liquids depending on the treatment. To operate the sprayer, press the nozzle at the top of the bottle with your index finger. Always spray from top to bottom from the forehead. Take all precautions to prevent the liquid from entering the nasal cavities. The client should keep eyes and mouth closed during the spraying operation. Keep an adequate distance from the face while spraying.

In both vaccination and spraying treatment, do not exceed 5 minutes. In case of overheating of the device, turn off all buttons and restart the device after 10 minutes. Once the treatment has been completed, switch off "POWER". Remove all tubes and accessories for disinfection.

Important!

Do not insert the sprayer tube into the vacuum socket to prevent any liquid from being sucked into the pump and damaging the unit. Do not operate the equipment for too long. Prolonged use may damage it.



Yubitzay Avila
Customer Training
T. +34 93 824 9020
y.avila@weelko.com
S. Yubitzay Avila

Weelko Barcelona S.
L.
Pol. Ind. La Valldan
Camí de Garreta, 92
08600 Berga
Barcelona – Spain
T. +34 93 824 9020
info@weelko.com
www.weelko.com